

RepliDens®

the endo education tool

03.2.1 - mandibular molar



| | |
|----------------------|--|
| level of difficulty: | medium - advanced |
| canals: | 4 |
| key feature: | 2 distal canals |
| working lengths: | mesiobuccal: 23.5 mm mesiolingual: 24 mm distobuccal and -lingual: 23 mm |
| working procedure: | use hand or rotary instrument system of choice* negotiate canals using an ISO 10 size hand file prepare glide path as indicated irrigate between instruments use water or alcohol for irrigation enjoy! |

*If you use a rotary instrument system, use a torque-controlled motor with settings provided by the manufacturer of the respective system.