
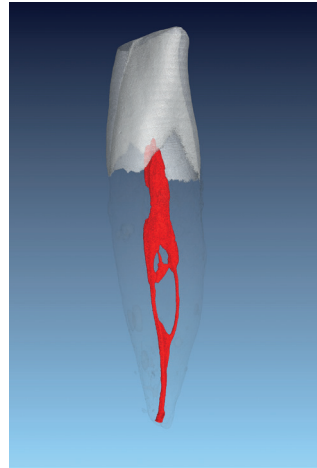


# RepliDens®

## the endo education tool

### 04.1.2 - mandibular incisor

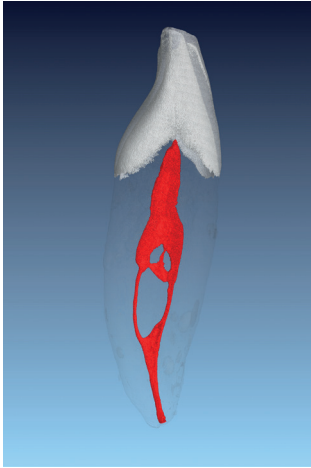

level of difficulty:	advanced
canals:	2
key feature:	no access cavity, Vertucci type II
working lengths:	buccal: 22 mm lingual: 22.5 mm
working procedure:	<p>use hand or rotary instrument system of choice*</p> <p>negotiate canals using an ISO 10 size hand file</p> <p>prepare glide path as indicated</p> <p>irrigate between instruments</p> <p>use water or alcohol for irrigation</p> <p>enjoy!</p>

\*If you use a rotary instrument system, use a torque-controlled motor with settings provided by the manufacturer of the respective system.