

RepliDens®

the endo education tool

01.1.1 - mandibular molar



level of difficulty:	moderate
canals:	3
key feature:	severely curved mesial canals
working lengths:	mesiobuccal: 23 mm mesiolingual: 23.5 mm distal: 21 mm
working procedure:	use hand or rotary instrument system of choice* negotiate canals using an ISO 10 size hand file prepare glide path as indicated irrigate between instruments use water or alcohol for irrigation enjoy!

*If you use a rotary instrument system, use a torque-controlled motor with settings provided by the manufacturer of the respective system.